

Instructions for use



Step 1 Cleanse wound bed and pat dry per protocol.



Step 7 Fill wound with saline-moistened antimicrobial gauze. Gauze should loosely fill to surface of wound. Avoid over packing.



Step 2 Apply skin sealant to periwound area.



Step 8 Place transparent dressing over the wound and seal.



Step 3 Trim non-adherent gauze and lay across wound bed.



Step 9 Create a seal by pinching the strip paste.



Step 4 Cut the drain 1" shorter than the base of the wound.



Step 10 Secure the drain tubing with Blue Tape®.



Curl the drain if using a channel or round drain.



Step 11 Connect the drain to the canister and begin suction.




Step 5 Apply layer of saline-moistened antimicrobial gauze to wound bed. Position drain on top of gauze.



Finished dressing Will have a "raisin like" appearance and be firm to the touch.



For channel drain, wrap layer of gauze around drain.

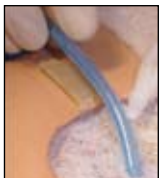

Step 6 Apply piece of ostomy strip paste to wound edge to secure drain in position.



Place second piece over top of drain and pinch in place.

Wound Management
Smith & Nephew, Inc.
11775 Starkey Road
PO Box 1970
Largo, Florida 33779-1970
USA

Customer Care Center
1 800-876-1261
T 727-392-1261
F 727-392-6914

www.smith-nephew.com
www.globalwoundacademy.com